|                                                                                  |          |
|----------------------------------------------------------------------------------|----------|
| <b>Instructions For Use</b>                                                      |          |
| <b>Bard® Surgical Felts, Pledgets and Tapes</b> . . . . .                        | <b>1</b> |
| Mode d'emploi                                                                    |          |
| Feutres, tampons et bandelettes chirurgicales Bard® . . . . .                    | 3        |
| Gebrauchsanweisung                                                               |          |
| Bard® Chirurgischer Filz, Nahtwiderlager (Pledgets) und Bänder . . . . .         | 5        |
| Instruzioni per l'uso                                                            |          |
| Felts (Feltri), Pledgets (Comprese di Garza) e Nastri Chirurgici Bard® . . . . . | 7        |
| Instrucciones de uso                                                             |          |
| Fieltros, algodoncillos y cintas quirúrgicas Bard® . . . . .                     | 9        |
| Gebruiksaanwijzing                                                               |          |
| Bard® chirurgisch vilt, tampons en band . . . . .                                | 11       |
| Instruções de utilização                                                         |          |
| Feltros, Compressas Pequenas e Fitas Cirúrgicas Bard® . . . . .                  | 13       |
| Οδηγίες χρήσης                                                                   |          |
| Χειρουργικά πιλήματα, επιθέματα και ταινίες Bard® . . . . .                      | 15       |
| Brugervejledning                                                                 |          |
| Bard® Kirurgisk Filt, Kompresser og Bånd . . . . .                               | 17       |
| Bruksanvisning                                                                   |          |
| Bard® kirurgiska patcher, kompresser och band . . . . .                          | 19       |
| Käyttöohjeet                                                                     |          |
| Kirurgiset Bard®-huovat, -tamponit ja -nauhat . . . . .                          | 21       |
| Bruksanvisning                                                                   |          |
| Bard® kirurgisk filt, kompresser og bånd . . . . .                               | 23       |
| Instrukcja użycia                                                                |          |
| Filce, tampony i taśmy chirurgiczne Bard® . . . . .                              | 25       |
| Használati utasítás                                                              |          |
| Bard® sebészeti filcek, tamponok és ragasztószalagok . . . . .                   | 27       |
| Pokyny k použití                                                                 |          |
| Chirurgický plstěný materiál Bard® - polštářky a pásky . . . . .                 | 29       |
| Kullanma Talimatı                                                                |          |
| Bard® Cerrahi Keçeler, Sargılar ve Yapıştırma Bantları . . . . .                 | 31       |

**BARD**

PERIPHERAL  
VASCULAR

## Bard® Surgical Felts, Pledgets and Tapes

### Instructions For Use

**Caution: Federal (U.S.A.) law restricts these devices to sale, by or on the order of a physician. Only physicians qualified in general surgical, vascular or cardiovascular techniques should use these products.**

#### Product Description:

- **Bard** Surgical Felts are constructed of either PTFE (polytetrafluoroethylene) or polyester (polyethylene terephthalate) fibers. They are available in a range of sizes, thickness, and permeabilities.
- **Bard** Pledgets are constructed of PTFE felt and are available in various sizes and shapes, including round, square, oval, and rectangle.
- **Bard** surgical tapes are made of either PTFE or polyester yarns and are tightly braided in construction.

#### Indications for Use:

- **Bard** PTFE and Polyester Surgical Felts are used in various applications for general, vascular and cardiac surgery. They are most commonly used as a patch, a buttress for sutures, and as a material for replacement of segments of the ventricular myocardium after resection.
- **Bard** PTFE Felt Pledgets are used as buttresses under sutures when there is a possibility of sutures tearing through tissue. These pledgets are used in various surgical suturing procedures, such as vascular closure, septal repair, myocardial closure, hepatic repair and valvular suturing.
- **Bard** PTFE and Polyester Surgical Tapes are indicated for the purposes of tissue approximation, retraction, and as a temporary or permanent ligature.

#### Contraindications:

None known.

#### Warnings:

1. **When used as a permanent ligature, care should be taken with surgical tapes to insure that the knots are secure.**
2. **Do NOT expose PTFE products to temperatures greater than 500°F (260°C). PTFE decomposes at elevated temperatures, producing highly toxic decomposition products.<sup>1</sup>**

#### Precautions:

1. STERILE, unless the package is opened or damaged. Single use only.
2. These products are sterilized by ethylene oxide. DO NOT RESTERILIZE.
3. Care should be taken when clamping surgical tapes to avoid damage to the braided material. Only atraumatic instruments should be used.
4. Taper point, noncutting needles are recommended when surgical tapes are used as an implant material so the fibers will not be cut during suturing.
5. Avoid excessive manipulation of the felt surface prior to use.
6. To assure that felt fibers are cleanly cut and not pulled apart, it is important to use a sharp cutting device or scissors with sharp blades when cutting felt in to smaller pieces.
7. After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

#### Potential Complications:

Potential complications that may occur with the use of these products or with any vascular implant procedure include infection and tissue erosion.

**Equipment Required:**

Non-resorbable suture with taper point needle  
Surgical instruments

**How Supplied/Stored:**

**Bard**<sup>®</sup> Surgical Felts, Pledgets and Tapes are supplied sterile, nonpyrogenic unless the package is damaged or opened. Single use only, do not resterilize. Pledgets (double pouched), tapes and felts (single pouched) are supplied 5-25 pouches per box. Refer to Warning #2 for storage conditions.

**Warranty**

Bard Peripheral Vascular warrants to the first purchaser of this product that this product will be free from defects in materials and workmanship for a period of one year from the date of first purchase and liability under this limited product warranty will be limited to repair or replacement of the defective product, in Bard Peripheral Vascular's sole discretion or refunding your net price paid. Wear and tear from normal use or defects resulting from misuse of this product are not covered by this limited warranty.

**TO THE EXTENT ALLOWABLE BY APPLICABLE LAW, THIS LIMITED PRODUCT WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT WILL BARD PERIPHERAL VASCULAR BE LIABLE TO YOU FOR ANY INDIRECT, INCIDENTAL OR CONSEQUENTIAL DAMAGES RESULTING FROM YOUR HANDLING OR USE OF THIS PRODUCT.**

Some countries do not allow an exclusion of implied warranties, incidental or consequential damages. You may be entitled to additional remedies under the laws of your country.

An issue or revision date and revision number for these instructions are included for the user's information on the last page of this booklet.

In the event 36 months have elapsed between this date and product use, the user should contact Bard Peripheral Vascular to see if additional product information is available.

**Reference:**

1. Guide to the Safe Handling of Fluoropolymer Resins, 3rd Edition, The Fluoropolymers Division of the Society of the Plastics Industry, Inc.

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**Bard is a registered trademark of C.R. Bard, Inc. or an affiliate.**
**Copyright © 2007 C. R. Bard, Inc. All rights reserved. Printed in the USA.**
Copyright © 2007 C. R. Bard, Inc. Tous droits réservés. Imprimé aux Etats-Unis.
Copyright © 2007 C. R. Bard, Inc. Alle Rechte vorbehalten. Gedruckt in den USA.
Copyright © 2007 C. R. Bard, Inc. Tutti i diritti riservati. Stampato negli Stati Uniti.
Copyright © 2007, C. R. Bard, Inc. Reservados todos los derechos. Impreso en EE.UU.