Instructions For Use

Bard® Parsonnet™ Pulse Generator Pouch	1
Mode d'emploi	
Poche pour générateur d'impulsions électriques Bard® Parsonnet™	3
Gebrauchsanweisung	
Bard® Parsonnet™ Tasche für Impulsgeneratoren	5
Istruzioni per l'uso	
Rivestimento per Generatore di Impulsi Bard® Parsonnet™	7
Instrucciones de uso	
Bolsa del generador de impulsos Bard® Parsonnet™	9
Gebruiksaanwijzing	
Bard® Parsonnet™ pulsgeneratorzakje	11
Instruções de utilização	
Bolsa do Gerador de Impulsos Parsonnet™ Bard®	13
Οδηγίες χρήσης	
Θήκη Γεννήτριας Παλμού Parsonnet™ της Bard®	15
Brugervejledning	
Bard® Parsonnet™ Pose til Pulsgenerator	17
Bruksanvisning	
Bard® Parsonnet™ påse för pulsgenerator	19
Käyttöohjeet	
Bard® Parsonnet™ -pulsigeneraattoripussi	21
Bruksanvisning	
Bard® Parsonnet™ pose til pulsgenerator	23
Instrukcja użycia	
Kieszon na generator impulsów Bard® Parsonnet™	25
Használati utasítás	
Bard® Parsonnet™ pulzusgenerátor zacskó	27
Pokyny k použiti	
Vak na pulzní generátor Bard® Parsonnet™	29
Kullanma Talimatı	
Bard® Parsonnet™ Nabız Atım Üretici Torbacığı	31

BARD

PERIPHERAL
VASCULAR

Bard® Parsonnet™ Pulse Generator Pouch

Caution: Federal (U.S.A.) law restricts this device to sale, by or on the order of a physician. Only physicians qualified in pulse generator implantation techniques should use this product.

Instructions For Use

Product Description:

The **Bard Parsonnet** Pulse Generator Pouch is constructed with polyester fibers. The knitted construction allows the pouch to stretch to accommodate various sizes and shapes of pulse generators.

Indications for Use:

The **Bard Parsonnet** Pulse Generator Pouch is designed to hold a pulse generator to create a stable environment when it is implanted in the body.

Contraindications and Warnings:

None known.

Precautions:

1. STERILE, unless the package is opened or damaged. Single use only.
2. These pouches are sterilized by ethylene oxide. **DO NOT RESTERILIZE.**
3. After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Adverse Reactions:

Adverse reactions that may occur with the use of these products or with any surgical procedure include bleeding or infection.

Equipment Required:

Scissors and suture.

Directions For Use

Insertion Technique


Remove the pouch from the package and place on a sterile field. Loosen the drawstrings of the opening of the pouch to facilitate insertion of the pulse generator. When the pouch covers all but the neck of the pulse generator, pull the drawstring tight. Tie the drawstring securely, using a square knot. Cut the excess drawstring and discard (see Precaution #3). The pulse generator is now ready for implantation, following standard accepted surgical techniques.

Large Pouches with Small Pulse Generators

In the event the pouch is too large for the pulse generator, the following method of tying is recommended:

After tying the drawstring, grasp the pouch at the opposite end and pull it tight so that the pouch is stretched firmly over the pulse generator. Then, one 360° twist will lock the pouch in position. Securely tie a nonabsorbable ligature at the point of the twist. Cut off the excess pouch and discard (see Figure 1 and Precaution #3).

Figure 1

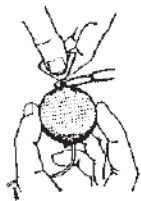Removal of Pulse Generator From Incorporated Pouch

It may be necessary to remove the pulse generator from the pouch after a period of implantation. First, surgically expose the pouch. Make an incision on the flat side of the pouch, approximately the width of the pulse generator. Disconnect the electrode leads. Remove the pulse generator through the opening in the side of the pouch. If required, insert a drainage tube. A new pulse generator may be inserted into the pouch through the side openings. Connect the electrical leads. Suture the pouch closed. Complete the procedure following standard, accepted surgical techniques.

WARRANTY

Bard Peripheral Vascular warrants to the first purchaser of this product that this product will be free from defects in materials and workmanship for a period of one year from the date of first purchase and liability under this limited product warranty will be limited to repair or replacement of the defective product, in Bard Peripheral Vascular sole discretion or refunding your net price paid. Wear and tear from normal use or defects resulting from misuse of this product are not covered by this limited warranty.

TO THE EXTENT ALLOWABLE BY APPLICABLE LAW, THIS LIMITED PRODUCT WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT WILL BARD PERIPHERAL VASCULAR BE LIABLE TO YOU FOR ANY INDIRECT, INCIDENTAL OR CONSEQUENTIAL DAMAGES RESULTING FROM YOUR HANDLING OR USE OF THIS PRODUCT.

Some states/countries do not allow an exclusion of implied warranties, incidental or consequential damages. You may be entitled to additional remedies under the laws of your state/country.

An issue or revision date and a revision number for these instructions are included for the user's information. In the event that 36 months have elapsed between this date and product use, the user should contact Bard Peripheral Vascular to see if additional product information is available.

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Bard and Parsonnet are trademarks and/or registered trademarks of C. R. Bard, Inc. or an affiliate.
Copyright © 2007 C. R. Bard, Inc. All rights reserved. Printed in the USA.
©2007 C. R. Bard, Inc. Alle rettigheder forbeholdes. Trykt i USA.
©2007 C. R. Bard, Inc. Eftertryck förbjudes. Tryckt i USA.
©2007 C. R. Bard, Inc. Kaikki oikeudet pidätetään. Painettu Yhdysvalloissa.
©2007 C. R. Bard, Inc. Med enerett. Trykt i USA.